

TECHNICAL LITERATURE STEEPER FOAM HANDS – FITTING INSTRUCTIONS

RSL Steeper Foam Hands are designed for cosmetic purposes only and are intended for use with RSL Steeper Cosmetic Gloves.

Note: A suitable handplate must be ordered separately

The hand is constructed with an integral "Round to Oval" interface which allows a round handplate to be mounted directly, using the self-tapping screws provided. The hand may therefore be fitted to a wrist disconnect unit and interchanged with a variety of functional devices.

CONVERSION TO AN OVAL WRIST

NOTE: This operation will reduce the length of the hand by about 5mm (3/16")

The interface (1) is attached to the core of the hand with four 16mm (5/8") long self-tapping screws (2). Remove these and the interface to expose the oval moulded base (3). Trim surplus soft foam square to the base with a sharp knife (4). A Steeper Oval Wrist Connector may then be fitted using the four 13mm (1/2") self-tapping screws provided with the hand. The hand moulding is sanded to give the oval wrist shape, taking care not to expose the core.

RSLSTEEPER, Products Division, Unit 7, Hunslet Trading Estate, Severn Road, Leeds, LS10 1BL, UK.
Tel: + 44 0870 240 4133 Fax: 44 0870 240 4779 email: sales@rehab.co.uk Web: www.rslsteeper.com

The design and manufacture of RSL STEEPER equipment and components are subject to a policy of continuous reappraisal. The Company therefore reserves the right to introduce changes and withdraw products without notice. © COPYRIGHT 2000, RSL STEEPER. Technical Literature No: L19536 issue 10 11 January 2001. €€

