



**TECHNICAL LITERATURE
SOFT PALP KITS**

Myo /Servo	2 ¼" Hand	2 ½" Hand	2 ¾" Hand	3" Hand	3 ¼" Hand
Left	E20509	E20511	E20429	E20431	E20433
Right	E20510	E20512	E20430	E20432	E20434

The finger and thumb palps which are fitted to the RSL Steeper Myo Electric and Servo Electric hands are retained in position by a stalk which passes through the body of the digit. In addition, they are glued in place to prevent peeling during glove change. Occasionally, the palps may require replacement.

To do this, first remove the damaged palp. Cut away any traces of the palp and adhesive. The exposed surface should be roughened with coarse glasspaper. Round any sharp edges. The replacement palp can then be fitted. Use Loctite™ 7070 cleaner, Loctite™ 454 gel adhesive to secure the palps. Trim away any of the stalk that is above the dorsal surface of the finger.

NOTE: Please follow the manufacturer's instructions for use and safety of primers and adhesives.

RSLSTEEPER, Products Division, Unit 7, Hunslet Trading Estate, Severn Road, Leeds, LS10 1BL, UK. Tel: + 44 0870 240 4133 Fax: 44 0870 240 4779 email: sales@rehab.co.uk Web: www.rslsteeper.com

The design and manufacturing of RSL Steeper equipment and components are subject to a policy of continuous reappraisal. The Company therefore reserves the rights to introduce changes and withdraw products without notice. © Copyright 2000, RSL Steeper. Literature No. L20669 Issue 10, 23 January 2001